

WATEARPROOF

Waterproof® Ear Protection
Class I MD - CE mark – Non sterile

Read this leaflet carefully before using this device. It contains important information for your treatment. If you have any further questions, ask your pharmacist or doctor.

1- What is WatEarProof®?

Waterproof® kit contains a sheet of polyurethane film and an adhesive gel tube for skin application.

The adhesive gel is composed of hypoallergenic silicone and ethyl acetate.

Ethyl acetate prevents the gel from hardening. This highly volatile ingredient has the characteristic odor of acetone-free solvent. Waterproof® adhesive gel is designed to be used exclusively on healthy skin.

The separate application of the gel – which closely adheres to the skin's contours - and the film provide genuine waterproofing and a good adhesion for up to 7 days. WatEarProof® protection maintains the ability to hear.

Its innovative concept forms a non-sticky layer over the protected area, allowing the skin to breathe and thus reducing the risk of maceration.

Do not use WatEarProof® protection for scuba diving.

2- What are the indications for using WatEarProof®?

WatEarProof® provides:

- Protection of the ear against water
- Protection of the ear against dust

WatEarProof® is transparent and sweat-resistant, enabling daily showers or baths.

Due to its waterproof and breathable properties, WatEarProof® reduces the risks of infection.

3- How to apply WatEarProof® protective system?

1. Apply a continuous thin line of adhesive gel all around the ear's perimeter.
 2. Wait for 1 minute without touching the gel to allow the solvent to evaporate.
 3. Fold in part of the card support attached to the polyurethane film. The transparent protection film appears
 4. Apply WatEarProof® over the ear and within the area outlined by the gel.
 5. Remove each side of the applicator frame to ensure proper adhesion between the adhesive gel and the film. Press along the gel line with your finger.
- You can trim any excess film beyond the gel line.

4- How to remove WatEarProof® protective system?

WatEarProof® can be removed after 4 hours of application. Removing it earlier may result in gel residue on the skin.

To remove the film and adhesive gel, gently pull the film.

5 - How long can WatEarProof® stay in place?

WatEarProof® can stay in place for up to 7 days.

6- What you need to know before using WatEarProof® protective system?

Do not use WatEarProof® in the following cases:

- Known allergies or reaction to silicone
- Known allergy or reaction to ethyl acetate or acetone-free solvents
- Children under the age of 36 months

Special warnings :

- Do not apply the gel to wounds, eyes, mouth, or mucous membranes.
- In the event of an allergic reaction, rinse thoroughly and contact your doctor immediately.
- Keep out of sight and reach of children.

Special precautions:

- Apply the gel only to healthy skin.
- Use a thin layer of gel.
- Do not use WatEarProof® as a primary dressing.
- Shave particularly hairy areas.
- File any calluses to aid adhesion.
- Wait at least 1 minute between applying the gel and the protective film to facilitate film removal.
- Wait at least 4 hours after applying the protective film before removing it to reduce gel residues.
- To remove the film and adhesive gel, gently and gradually pull on the film.

7-How to store WatEarProof®?

Store away from light, heat, and moisture.

Further information

Manufactured by: SAS DOUCHOSSEC – SECUDERM,

Galerie Voltaire, Avenue Roger Salengro, 13400 AUBAGNE, France

Imported by: Drywaters limited, 17 Bay Lair Grove, Island Bay, 6023 Wellington

Date of last leaflet revision: 02/2025



Flammable



Do not re-use



Consult instructions
for use



Keep away
from sunlight



Keep dry



High strength